



# ADC COMPACT PREVIEW/ ID STATION



Place the cassette in the ID TABLET (The RF Tag should be at upper right hand corner). Slide Tag under green light.

At prompt, type in **ACCESSION NUMBER** and press **ENTER**, use Bar Code Scanner, or manually enter patient demographic data.



Check Patient Demographics. Very Important to Prevent Broken Studies!!!!!!!!!!!!

**SELECT Radiologist (Adult or Peds)**

**SELECT Examination (Body Part)**

**SELECT Sub-examination**

**(Position or Projection)\***

**VERIFY Patient Position (AP) \* (RT, LAT) \* (PA) (LT, LAT)**

**VERIFY CASSETTE ORIENTATION: Landscape OR Portrait**

**TYPE any comments in Comment Section**

**If all is correct, CLICK ON F1 OR PRESS F1**

Screen will display "PLEASE REMOVE CASSETTE". Place the cassette in DIGITIZER!!!!!!!!!!!!!!

**\*NOTE\*:** Once Sub-Examination is selected, **Patient Position, Cassette Orientation, and Exposure Class** will **DEFAULT** to the selected exam. At this point, only verification is required.

**F1 WRITE:** writes the patient demographics to the cassette

**F2 CLEAR:** clears the information from each text box

**F3 RECALL:** continuously recalls the last 20 cassettes/patients that were identified

**F4 NEW PATIENT:** clears the patient demographics section of the screen to allow new patient data to be entered

**F7 HISTORY:** presents a list of the last 20 cassettes/Patients identified

**F10 CANCEL:** cancels the ID STATION WINDOW and asks you to remove the cassette