



AGFA

ADC COMPACT PROCESSING STATION (PS)

- MOST RECENTLY PROCESSED IMAGE WILL APPEAR AT UPPER LEFT BOX *IF*:

- Screen is on Page 1
- Pull-down lists in the upper left-hand corner should all have a *
- SORT ON box should read "Identification date"
- Controller should read "Image Processing"

- **HIGHLIGHT** the desired image(s). Ten (10) images can be selected at once.

- To **DISPLAY** image(s), click on OPEN (located at upper right portion of screen) to view image. If visualization of one image is desired, double click on box.

- **CRITIQUE IMAGE** (W/L, FLIP, ANNOTATE, ADD MARKERS, ETC.)

- *NOTE: Always flip image FIRST before annotation or markers are added, otherwise annotation and markers will be deleted from image)*



- Click on **NEXT** and **PREVIOUS** to navigate through images

- Check **INFO BOX**:

- ◆ Validate patients demographics
- ◆ Make any changes to patient demographics (white text only)
- ◆ Check the **EXPOSURE LEVEL (LGM)**
- ◆ An optimal LGM reading should be between 2.0 and 2.2.
- ◆ Any values above and below those numbers indicate an overexposure or underexposure.



- **Changes to image(s) MUST BE SAVED:**

- ✓ Click on SAVE icon
- ✓ To replace image with updated changes, click IN PLACE
- ✓ To retain original data AND save a new image, click AS NEW
- ✓ For saving a single image, click SAVE
- ✓ For saving multiple images, click ALL

- **After ALL changes have been made, send images to IMPAX:**

- Click on the ARCHIVE icon
- Click on the Destination
- Click on ARCHIVE to send one image
- Click on ALL to send multiple images



- **TO PRINT IMAGES:**

- Select Film size
- Select Layout (Landscape or Portrait)
- Select Film usage (always select TRUE SIZE 1:1)
- Select Printer
- Click on PRINT



- **Verify Patient Demographics at BOTTOM of screen**
- To return to the Main Browser Screen, click on **SELECT**.

